



AMD Patient Camera General Examination

Start-up and Operation Procedures

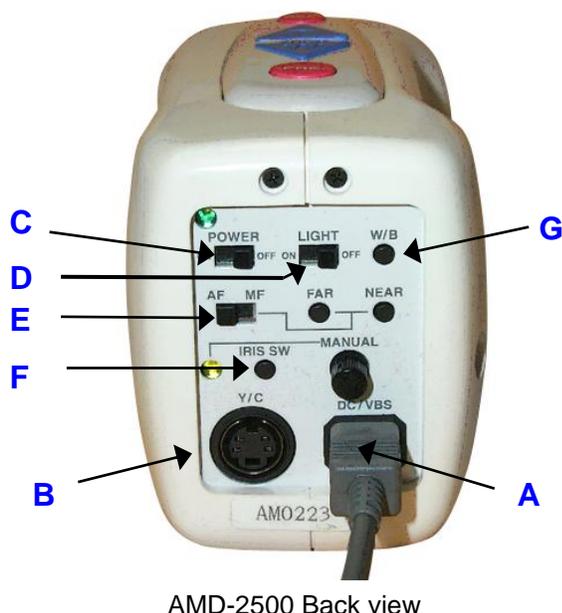

AMD-2500 Back view

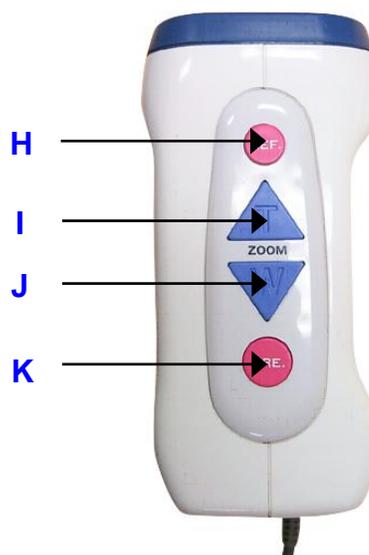

AMD-2500 Top view

Start-up Procedure

1. Plug in **Grey DC/VBS** cord (A).
2. Plug in **Black Y/C** cable (B). (*Note:* Some systems do not require the black v/c cable due to wiring through a switcher box)
3. Turn **POWER** switch to "ON" (C).
4. Turn **LIGHT** switch to "ON" (D).
5. Set **AF/MF** switch (E) to "AF" (autofocus). (*Note:* Recommended setting: Autofocus)
6. Be sure that **IRIS SW** light (F) is "OFF".
7. Point the camera at a blank sheet of white paper and hold approximately 10 cm away. Press and hold the **W/B (white balance)** button (G) for 3 seconds then release. (*Note:* turn camera light on for white balance)
8. Zoom all the way out by holding down the blue **W** (wide angle) button (J).

Operation Procedure

9. To better view surface structures, press the red **REF** (reflection) button (H) to turn the light polarization "OFF".
10. Zoom in at small increments by pressing the blue **T** (telephoto) button (I).
11. Press **W** button (J) to zoom out to view the image at sufficient distance to show a good overall view.
12. Freeze the image by pressing the red **FRE** button (K). To unfreeze press **FRE** button again.

Using the AMD Camera

Tips for using the Patient Exam Camera

- ✓ Use a tripod to help to help stabilize the camera image especially when showing close up area
- ✓ Always white balance the camera to reset the colors
- ✓ Keep the camera in autofocus position (AF)

Troubleshooting for the AMD Patient Camera

- Blurry Images: Try using a tripod to stabilize the image especially when zooming in on a close-up area
- Black and White Pictures: Check the prongs on the s-video cable that connects the patient exam to the videoconference system. If a prong is bent the cable will need to be replaced.
- Imbalanced: White Balance the camera every time you use it by placing it 3 ' above a white piece of paper and holding the W/B button found at the back of the camera.
- Frozen Image: If an image appears frozen on the screen press the FRE button on the top to unfreeze the image.
- When using a surgical glove tips for magnification lens simply remove and wipe probe with alcohol wipe.
- Clean the camera surface with a soft cloth with alcohol or mild detergent
- Clean the regular lens using an eyeglass cleaner.